


FOREHEAD PAD REPLACEMENT

If you need to replace the forehead pad, contact your home care provider to reorder. Follow the instructions below.

Step 1 Remove the pad. Slide the pad outward toward one edge of the forehead support and lift it upward so the connection track on that side is released from the plastic connection slot. Next, pull the pad in the opposite direction to release the connection track on the other side from its connection slot.

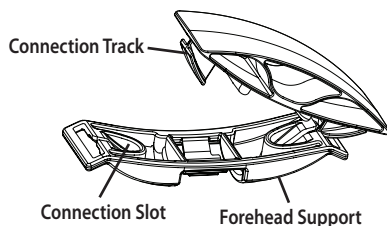

Step 2 Install the new pad. Slide one of the connection tracks on the new pad into its connection slot on the support. Pull the pad in the opposite direction (toward the center) and slide the other connection track into its connection slot to lock the pad into place.

NOTE: Replace the cushion and forehead pad if damaged, deformed, or hardened.


DISPOSAL OF PARTS

Dispose of any damaged or broken parts in accordance with local regulations.

REORDER NUMBERS

1002800 Deluxe Headgear	1018401 Forehead Support and Pad
1007933 ComfortSelect Mask, S	1009722 Silicone Forehead Pads, 4 pack
1007932 ComfortSelect Mask, M	1009723 Headgear Swivel Clips (2 pk)
1007934 ComfortSelect Mask, Small Wide	1008814 ComfortSelect Retaining Ring
1007930 ComfortSelect Mask w/hgr, S	1004395 Exhalation Elbow Kit
1007919 ComfortSelect Mask w/hgr, M	1002720 Split Washer Kit (5 ea.)
1007931 ComfortSelect Mask w/hgr, Small Wide	7041 Mask Swivel
1007936 ComfortSelect Cushion, S	312710 O ₂ Enrichment Attachment
1007935 ComfortSelect Cushion, M	
1007937 ComfortSelect Cushion, Small Wide	

Visit www.respironics.com for a complete list of products and reorder numbers, or contact Customer Service at 1-800-345-6443 (US or Canada only) or 724-387-4000 to obtain a product catalog (#1027825). Visit www.comfortseries.respironics.com for information about the ComfortSelect and other Comfort Series masks.

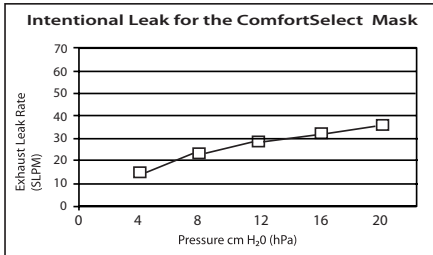

Deadspace	Pressure Drop cm H ₂ O (hPa)	
	50 LPM	100 LPM
S 119 cc		
M 129 cc		
SW 128 cc		
	All Cushion Sizes:	
	S-M-SW	0.1 0.2

Storage: Temperature -4° F to 140° F (-20° C to 60° C)
Relative Humidity up to 95%, non-condensing



1001 Murry Ridge Lane
Murrysville, PA 15668

www.comfortseries.respironics.com
Respironics, Comfort Series, and ComfortSelect are registered trademarks of Respironics, Inc. and its affiliates. Covered by one or more of the following patents: 6,851,425 Des. 483,477, Des. 489,817. Other Patents Pending. © 2006 Respironics, Inc. and its affiliates. All rights reserved. Latex free.

1033263
JW 09/17/06



ComfortSelect™


INSTRUCTIONS FOR USE

MULTI-PATIENT USE REUSABLE

LATEX FREE

INTENDED USE

The ComfortSelect Nasal Mask and Headgear are intended to provide an interface for adult patients when used with Respironics CPAP or bi-level therapy. The mask contains an exhalation device that provides a continuous leak path in the patient circuit. This mask does not require the use of a separate exhalation device.

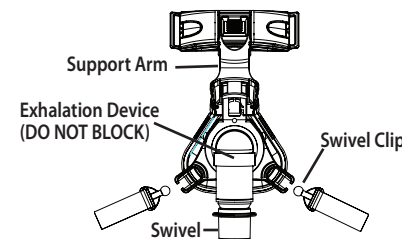

CAUTION!

US federal law restricts this device to sale by or on the order of a physician.

SYMBOL



Caution, consult accompanying documents.

WARNINGS

- This CPAP mask should be used only with CPAP or bi-level systems recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or bi-level system is turned on and operating properly. The vent hole or holes associated with the mask should never be blocked. **Explanation of the Warning:** CPAP and bi-level systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP or bi-level machine is turned on and functioning properly, new air from the CPAP or bi-level machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP or bi-level machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This warning applies to most models of CPAP and bi-level systems.
- At low CPAP pressures, the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing

IMPORTANT!



Hand wash mask before use.

- may occur.
- At a fixed flow rate of supplemental oxygen flow the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection and the leak rate. This warning applies to most CPAP or bi-level systems.
- If oxygen is used with the CPAP or bi-level system, the oxygen flow must be turned off when the system is not in operation. **Explanation of the Warning:** When the CPAP or bi-level system is not in operation and the oxygen flow is left on, oxygen delivered in the ventilator tubing may accumulate within the CPAP or bi-level system enclosure. Oxygen accumulated in the CPAP or bi-level system will create a risk of fire. This warning applies to most CPAP or bi-level systems.
- Oxygen accelerates fires. Keep the CPAP or bi-level system and oxygen container away from heat, open flames, any oil substance, or other sources of ignition. Do not smoke in the area near the device or the oxygen container.
- Some users may experience skin redness. If this occurs, consult your healthcare professional.

IMPORTANT! Hand wash mask before use.

Before use clean the mask following the instructions listed below.

CLEANING INSTRUCTIONS

For Multi-Patient Use Reusable. Contact Respironics Customer Service at 1-800-345-6443 to obtain Mask Disinfecting Instructions (order #1011138).

For Home Use: Begin by removing or disconnecting the headgear from the mask. Clean the headgear at least once each week or more often if needed. Fasten the hook tabs on the straps so that they don't catch other garments in the laundry. Some Respironics' masks allow the cushion to be disassembled from the faceplate. The user has the option of disassembling the cushion from the faceplate before cleaning.

Faceplate and Cushion

Thoroughly clean the mask each day, immediately after use, using the hand wash instructions.

Hand Wash Instructions






- Step 1** Hand wash the mask in warm water with a mild dishwashing detergent. **Do not** use bleach, alcohol, cleaning solutions containing alcohol, or any strong household cleaners. **Do not** use cleaners containing conditioners or moisturizers.
- Step 2** Rinse thoroughly. Air dry. Make sure the mask is dry before use. Inspect the mask after cleaning.
- Replace the mask if any parts are damaged or deteriorated.**

Headgear with Swivel Clip Mask Accessories

- Step 1** Hand or machine wash the headgear and attached swivel clips with a standard laundry detergent and warm water. Rinse thoroughly. **Do not** use bleach.
- Step 2** Line dry or machine dry at the medium heat setting.
- NOTE** Slight readjustment of the straps may be needed after repeated washings.

PUTTING ON THE HEADGEAR AND MASK

- Step 1** Hold the mask cushion gently over your nose. If the forehead support does not rest comfortably against your forehead, press the ridged sides at the bottom of the support arm, and slide the arm up or down the bar until the arm is in a position that is comfortable for you. Slide the headgear over your head. You may want to disconnect one of the swivel clips before putting on the headgear. Reconnect the swivel clip after you have the headgear in position.
- Step 2** Connect the flexible tubing (included with your CPAP or bi-level device) to the swivel on the mask.
- Step 3** Turn on the airflow.
- Step 4** Lie down and breathe normally through your nose, keeping your lips closed. You may need to adjust the forehead support arm to reduce leaks or reduce pressure points at the bridge or above the lip.
- Step 5** Unhook the straps and gradually tighten them until the mask has minimal leaks and fits comfortably.

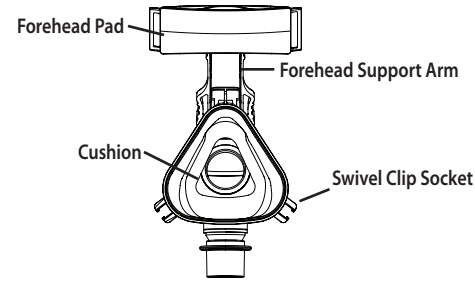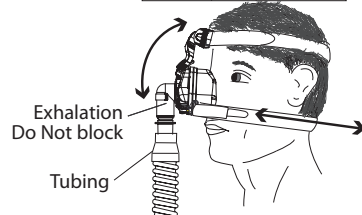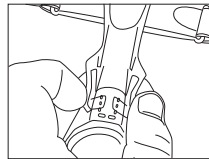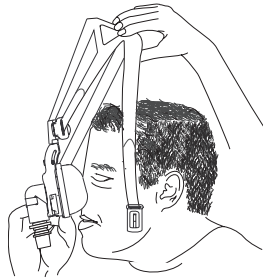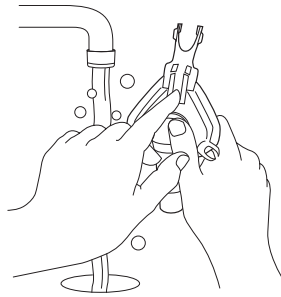


Helpful Tips for Mask Comfort

- Clean your face and mask thoroughly before each use.
- Make sure the mask is the correct size for your face.
- **Do not** overtighten the headgear. Overtightening can irritate your face, increase leaks, or cause damage to the mask.
- Replace the cushion and forehead pad if they become hardened or deformed.


REMOVING THE HEADGEAR AND MASK

You can disconnect the headgear from the mask in the following ways:


- Disconnect the forehead support by pressing on the center of the forehead support and then gently pressing the sides of the forehead support arm together.
- Disconnect the neck straps by removing the swivel clips from their sockets.
- Slide the headgear over your head. The entire headgear stretches.

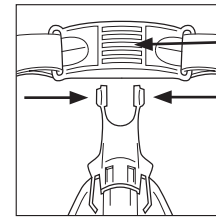

a) Press center of forehead support -then- Press sides of support arm together

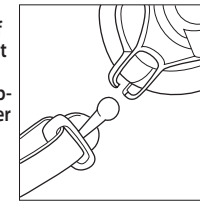

b) Remove swivel clip from socket

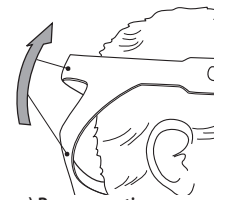


c) Remove entire assembly as you would a cap

REPLACEMENT OF PARTS

Use the reorder list on the next page to order replacement parts. Follow the instructions below to replace the items.

CUSHION REMOVAL AND REATTACHMENT

Remove the cushion by detaching its retaining ring from the faceplate. Do this by unhooking the side and bottom clips of the ring from their connection points on the faceplate. Take the ring off of the cushion. Clean or discard the cushion if necessary. Place the cleaned or new cushion on the faceplate, making sure that the two slits at the bottom of the cushion catch on and fit securely into the two tabs at the bottom of the faceplate. Place the retaining ring around the cushion. Connect the ring clips to the faceplate, starting with the top clip. Push the retaining ring into the faceplate until you hear a clicking sound.

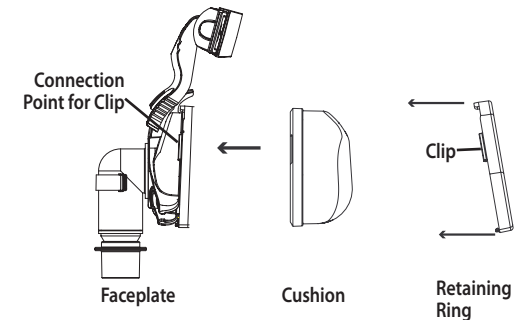

NOTE: Make sure the lines on the bottom of the faceplate, cushion, and retaining ring are all aligned when assembled.