

Cleaning and Disinfecting the ComfortFull™ 2 Mask

CLEANING

Clean the mask prior to using any of the disinfection or sterilization methods listed below.

- Remove the headgear and spacer, if appropriate.
- Soak the parts in a commercially available enzymatic cleaner (e.g., Klenzyme™).
- Clean the mask with a soft bristle brush in a solution of cool tap water and a commercially available anionic detergent (e.g., Manu Klenz™). Do not use cleaning products that contain conditioners or moisturizers because they will leave a residue.
- Rinse thoroughly under cool running tap water. Air dry.
- If adhering substances have not been adequately removed, replace the mask.



DISINFECTION/STERILIZATION PROCESS

PASTEURIZATION

TEMPERATURE	EXPOSURE TIME	RESULT	NO. CYCLES TESTED	NOTES
70° C (performance tested at 75° C)	30 minutes	High level disinfection	30	<ul style="list-style-type: none"> ▪ Inspect the mask after processing. If any components are damaged, replace the mask. ▪ Slight discoloration of the mask cushion after processing is normal.
90° C	1 minute			

CIDEX 2.4%

EXPOSURE TIME	RINSE	RESULT	NO. CYCLES TESTED	NOTES
45 minutes	Per manufacturer's instruction	High level disinfection	30	<ul style="list-style-type: none"> ▪ Inspect the mask after processing. If any components are damaged, replace the mask. ▪ Slight discoloration of the mask cushion after processing is normal.

CIDEX O.P.A.

EXPOSURE TIME	RINSE	RESULT	NO. CYCLES TESTED	NOTES
12 minutes	Per manufacturer's instruction	High level disinfection	30	<ul style="list-style-type: none"> ▪ Inspect the mask after processing. If any components are damaged, replace the mask. ▪ Slight discoloration of the mask cushion after processing is normal.

WAVICIDE®

EXPOSURE TIME	RINSE	RESULT	NO. CYCLES TESTED	NOTES
45 minutes	Per manufacturer's instruction	High level disinfection	30	<ul style="list-style-type: none">▪ Inspect the mask after processing. If any components are damaged, replace the mask.▪ Slight discoloration of the mask cushion after processing is normal.

Hydrogen Gas Plasma Sterilization (STERRAD®)

EXPOSURE TIME	RINSE	RESULT	NO. CYCLES TESTED	NOTES
Machine Controlled	N/A	Sterilization	30	<ul style="list-style-type: none">▪ Inspect the mask after processing. If any components are damaged, replace the mask.▪ Slight discoloration of the mask cushion after processing is normal.

Contact Customer Service to obtain the most current mask disinfecting instructions for the ComfortFull 2 (#1026814).

Contact Customer Service to obtain mask disinfecting instructions for other Respironics masks (#1011138).

Customer Service: 1-800-345-6443 • 724-387-4000

Respironics Europe: +33 1 47 52 30 00

Respironics Asia Pacific: +81-3-5280-9611


RESPIRONICS®
1001 Murry Ridge Lane
Murrysville, Pennsylvania
15668-8550 USA

RESPIRONICS®
Deutschland
Gewerbestrasse 17
82211 Herrsching Germany



©2006 Respironics, Inc., and its affiliates. All rights reserved.

1031389
JW 12/29/05