

# Citrus II®

## A Fresh Approach for Cleaning Tubing and Accessories



Citrus II CPAP Mask Wipes  
Contains 62 Wipes  
5" x 8" (12.7 cm x 20.3 cm)  
# 635871639

Citrus II CPAP Mask Cleaner and Citrus II CPAP Concentrated Cleanser contain a unique formula designed for the effective cleaning of all types of CPAP and BIPAP masks, nasal pillows, nasal cannulas or O2 Masks.

Using all natural ingredients made from vegetable-derived cleansers and solvents, the Citrus II CPAP cleaners gently remove dirt, oils and organic matter from masks and tubing.

The mask cleaners are designed for everyday use and are available in 62 count wipes, 1.5 oz. non-aerosol travel spray and an 8 oz. non-aerosol spray.

The concentrated cleanser is designed for weekly cleaning of tubing and accessories. Simply mix 4 oz. of concentrate with 1 gallon of water, soak the tubing for 15 to 20 minutes, rinse and air dry.



1.5 oz. Spray  
# 635871164



8 oz. Spray  
# 635871165



Citrus II CPAP Concentrated Cleanser  
32 oz. Bottle  
Makes 8 Gallons  
# 635871849



20-Piece Display Contains:  
9 - CPAP Mask Wipes, 7-1.5 oz. Spray Cans, 4-CPAP Concentrated Cleansers  
(16.5"H x 17.5"W x 10.75"D)  
# 236371333

SS #63026 REV 0508

**Beaumont Products, Inc. • Kennesaw, GA 30144**  
**1-800-451-7096 • Fax 770-514-7400 • www.citrus2.com**