

ComfortFusion™


Instructions for Cleaning and Disinfection

CAUTION! For Clinical Use Only. The following processes are validated for this mask when used in accordance with the cleaning and disinfection manufacturer's instructions. Any deviation from the manufacturer's instructions or agents listed below may impact the performance or durability of the product. See manufacturer's instructions for additional warnings and cautions.

WARNING! Inspect the mask components after disinfection and replace any components that are damaged. See the *Instructions for Use* included with the product for additional warnings and cautions.

Notes

- The headgear cannot be processed using the disinfection methods listed. The headgear must be replaced or washed before reuse. See the *Instructions for Use* included with the product.
- After disinfection, discoloration of the cushion is normal.

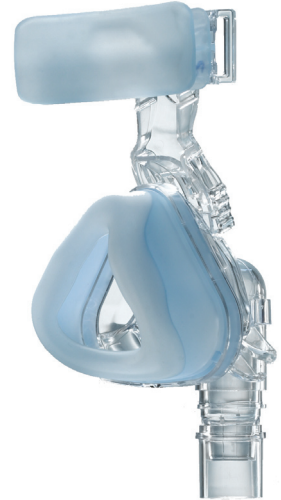

CLEANING

- Disassemble the mask per the *Instructions for Use* included with the product.
- Use a soft bristle brush to adequately remove adhering substances from each component while soaking in a commercially available, anionic detergent such as Manu Klenz® or MediZyme®. Extra attention should be given to the crevices and cavities during cleaning.
- Rinse the mask and components with potable water (minimum 5 liters per mask) and air dry out of direct sunlight.

DISINFECTION

The following disinfection processes are intended for multi-patient use in a clinical or hospital environment only. For home cleaning or single patient use, refer to the *Instructions for Use* included with the product.

Thermal Disinfection: Use only one option

TEMPERATURE	TIME	NUMBER OF CYCLES TESTED
75° C	30 minutes	30
or		
90° C	1 minute	30

Chemical Disinfection: Use only one option

TREATMENT	EXPOSURE TIME	SOLUTION	RINSE	NUMBER OF CYCLES TESTED
CIDEX® Activated Dialdehyde Solution	45 minutes	Per Manufacturer's Instructions	Per Manufacturer's Instructions	30
or				
CIDEX OPA®	12 minutes	Per Manufacturer's Instructions	Per Manufacturer's Instructions	30
or				

TREATMENT	EXPOSURE TIME	SOLUTION	RINSE	NUMBER OF CYCLES TESTED
WAVICIDE®	45 minutes	Per Manufacturer's Instructions	Per Manufacturer's Instructions	30
or				
CONTROL III™	10 minutes	Per Manufacturer's Instructions	Per Manufacturer's Instructions	30

Hydrogen Peroxide Gas Plasma

TREATMENT	EXPOSURE TIME	NUMBER OF CYCLES TESTED
STERRAD®	Machine Controlled	30

English only, REF 1011138. For additional languages visit www.respironics.com, or contact Respironics and state which language or languages are needed. You may also request a product catalog (REF 1027825) for a complete list of products and reorder information.

Respironics: 1-800-345-6443 (USA or Canada only) or 724-387-4000
 Respironics Deutschland: +49-8159-93060

RESPIRONICS®

EC REP

1001 Murry Ridge Lane
 Murrysville, PA 15668
www.respironics.com

Respironics Deutschland
 Gewerbestrasse 17
 82211 Herrsching, Germany

CE 0123

ENGLISH

Respironics is a registered trademark, and ComfortFusion is a trademark, respectively, of Respironics, Inc. and its affiliates. CIDEX, CIDEX OPA and STERRAD are registered trademarks of the Advanced Sterilization Products. Wavicide is a registered trademark of Medical Chemical Corporation (MCC). Control III is a trademark of Maril Products, Inc. Manu Klenz is a trademark of Steris Corporation. MediZyme is a registered trademark of MEDICHEM International Ltd. © 2007 Respironics, Inc. and its affiliates. All rights reserved. Latex free and Di(2-ethylhexyl)phthalate (DEHP) free.

REF 1011138
 1042047
 JW 10/26/07