

# Don't lose a wink of sleep this Daylight Savings



**It's that time again...**

**... to adjust the clock on your F&P ICON™ for Daylight Savings**



**1.** Turn the SmartDial™ to the Clock Time Setting and press to enter.



**2.** Turn the SmartDial™ to select the hour, minutes, 12-hour or 24-hour time.



**3.** After each selection, press again to accept the change.

**... and refresh your equipment**

Equipment Checklist	Good	Needs replaced
Mask	<input type="checkbox"/>	<input type="checkbox"/>
Headgear	<input type="checkbox"/>	<input type="checkbox"/>
Tubing	<input type="checkbox"/>	<input type="checkbox"/>
Water Chamber	<input type="checkbox"/>	<input type="checkbox"/>
Air Filter	<input type="checkbox"/>	<input type="checkbox"/>

Contact your Homecare Provider below



**Stylish on the Outside.  
Smart on the Inside.**